

Impression lacquer

for sealing and smoothing
of silicone impressions
silicone based liquid polish
cold curing

Instructions for use

Storage: Keep tightly closed at temperature not exceeding 25 °C/77 °F.

Application: Prior to applying liquid polish the impression must be absolutely clean and dry!

Clean the surface of the impression with an appropriate cleaner and then apply the lacquer (dipping or brush on).

After dripping-off let the impression vulcanize in the air for approx. 2-3 hours.

The surface of the silicone impression exhibits now a glossy layer.

Caution: Avoid contact with eyes and skin. Do not breathe vapour.

Important: Close containers carefully after each use!

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 • 76275 Ettlingen/Germany
Telefon: 07243/510-0 • Fax: 07243/510-100
www.detax.de • mail@detax.de

