

detax addition

Instructions for use

***Ear impression material, A-silicone based,
lasting dimensional stability***

Instruction

for use:

Remove impression material A (white) and B (blue) using measuring scoops. Mix together (mixing ratio 1:1), until a uniform colouring is achieved. Fill the homogeneous mixture into an impression syringe and apply it slowly and bubblefree into the ear canal.

Mixing temperature: approx. 23 °C/73 °F

Mixing time:

approx. 30 - 45*sec.

Working time:

approx. 1 min. 30 sec. - 2 Min.*

Setting time:

approx. 5 min. - 6 min.* according to mixing time

* from beginning of mixing

Increased temperatures accelerate, decreased temperatures retard the setting time.

Caution!

Avoid mutual contact of components A+B before mixing. Do not interchange measuring scoops and lids. Do not remove material from container by hand. Do not use latex-gloves.

Storage:

Keep containers closed and at room temperature (approx. 23°C/73°F) - protect from light and heat.

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 • 76275 Ettlingen/Germany
Telefon: 07243/510-0 • Fax: 07243/510-100
www.detax.de • post@detax.de

