

Ear impression material, C-silicone based

Storage and at approx. 23°C/73°F, 50% +- rel. humidity,

application: light protected

Colour code: red

Aroma: lemon

Final hardness: approx. 25 Shore A

Dosage: impression material = 1 measuring scoop filled to the top + 6 cm of catalyst paste

Application: Put putty onto the hand and flatten out. Press rim of measuring scoop into putty. Apply catalyst paste along impression of measuring rim up to the marking (=6 cm). Knead well, until an uniform colouring is achieved.

Mixing time: approx. 30 - 45* sec.

Setting time: approx. 5 min. - 6 min.* depending on mixing time and amount of catalyst paste

Caution ! Cured materials are chemically inter-spots on clothing should be avoided.

DETAX

GmbH & Co. KG
Carl-Zeiss-Str. 4 • 76275 Ettlingen/Germany
Telefon: 07243/510-0 • Fax: 07243/510-100
www.detax.de • post@detax.de