

*Vinylpolysiloxane - very high consistency,
with „snap set effect“*

Mixing

temperature: approx. 23 °C/73 °F

Instruction

for use: Remove impression material A (yellow) and B (green) using measuring scoops. Mix together (mixing ratio 1:1), until a uniform colouring is achieved. Fill the homogeneous mixture into an impression syringe and apply it slowly and bubblefree into the ear canal.

Mixing time: approx. 30 sec.

Working time: approx. 2 min.*

Setting time: approx. 4 min. 30 sec. - 5 min.*
at body temperature

* from beginning of mixing

Increased temperatures accelerate, decreased temperatures retard the setting time.

Caution: Avoid mutual contact of components A+B before mixing. Do not interchange measuring scoops and lids. Do not remove material from container by hand. Do not use latex-gloves.

DETAX

GmbH & Co. KG
Postfach 10 02 25 • 76256 Ettlingen/Germany
Telefon: 07243/510-0 • Fax: 07243/510-100
www.detax.de • mail@detax.de

